



EntericBio<sup>®</sup>

## Bacteria

- EntericBio Bacterial Panel 2 RUO

The EntericBio Molecular system is rapid, reliable and easy to use, providing results within 3 hours direct from faecal samples.

The system can process 1-48 samples per run with no DNA extraction, automated setup and lyophilised ready to use assays.



# EntericBio realtime<sup>®</sup>

CE IVD EntericBio Bacterial Panel 2 RUO

VT1

VT2

## VT1

Reference Method	Total	EntericBio	
		Positive	Negative
Positive	40	40	0
Negative	4	4	502

- Sensitivity 1.00 (0.89-1.00, 95% CI)
- Specificity 0.99 (0.98-1.00, 95% CI)
- PPV 0.91 (0.77-0.97, 95% CI)
- NPV 1.00 (0.99-1.00, 95% CI)

## VT2

Reference Method	Total	EntericBio	
		Positive	Negative
Positive	44	44	0
Negative	1	1	504

- Sensitivity 1.00 (0.90-1.00, 95% CI)
- Specificity 0.99 (0.99-0.99, 95% CI)
- PPV 0.98 (0.87-0.99, 95% CI)
- NPV 1.00 (0.99-1.00, 95% CI)

EntericBio



### Serosep Limited

Annacotty Business Park  
Annacotty, Limerick,  
Ireland

Telephone +353 (0)61 358 190

Email info@serosep.com

### Serosep UK Limited

14 The Brunel Centre  
Newton Road  
Crawley, West Sussex  
RH10 9TU, UK

Telephone +44 (0) 129 388 9215